

Quick Guide


Ambient Lighting (MPN: EXT-AMD-0001)

For further information, please see user manual.

OPASCA GmbH
Franz-Volhard-Str. 3
68167 Mannheim
Germany

Phone: +49 (0) 621 877537-10
Email: contact@opasca.com
opasca.com

Select the Ambient Lighting settings on the Cockpit Terminal

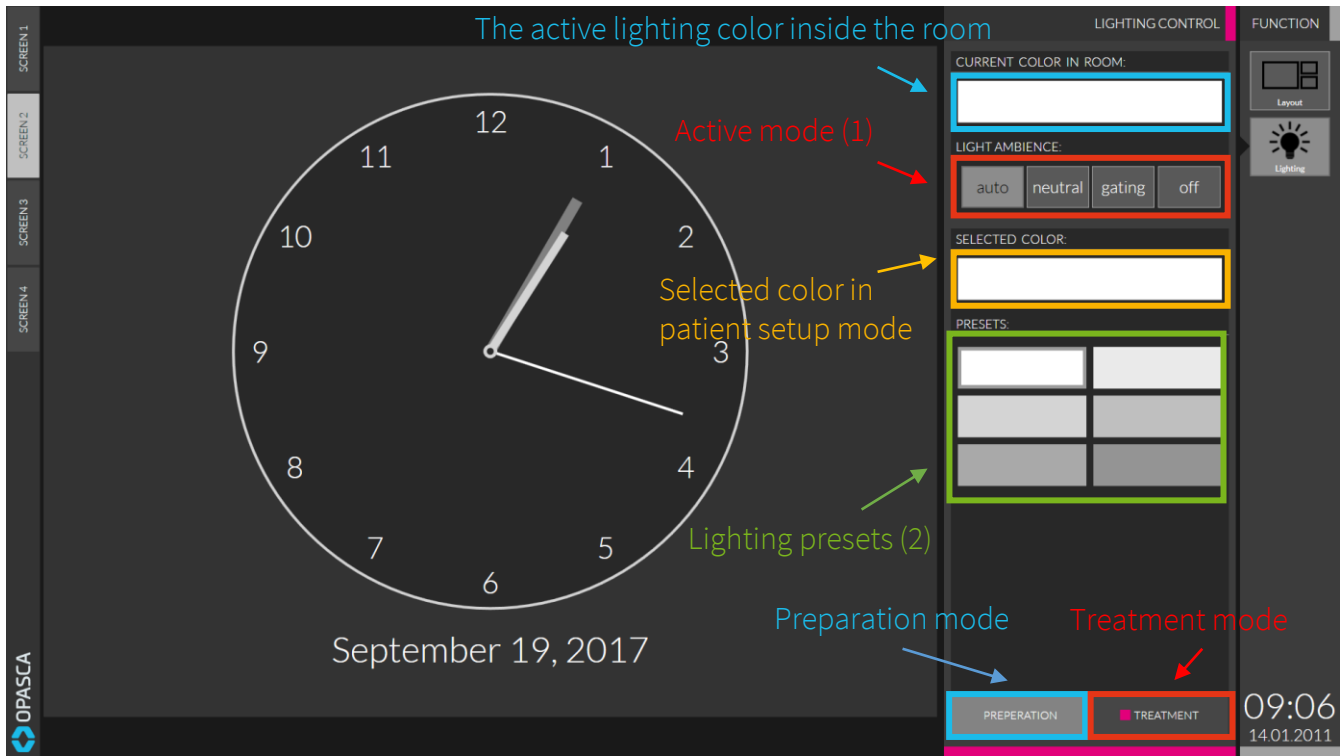

(1) Lighting-Modes

auto - The light automatically switches between set-up and treatment mode when the radiation protection door is opened and closed.

neutral - Constant neutral (white) lighting is set

gating - The lighting is used for gating feedback

off - The lighting system is disabled


(2) Lighting-Settings

Select from a set of predefined configurations or set new pre-sets.

Important:

- The mode “**gating**” is only available in combination with the Feature Gating Feedback (EXT-GAF-0001 und EXT-GAF-0002) and a connected gating system (C-RAD Catalyst® or vision RT AlignRT®).
- The mode “**auto**” switches automatically, depending on the interlock state between preparation (neutral white lighting) and treatment (Ambient Lighting).

Saving Ambient Lighting settings as presets

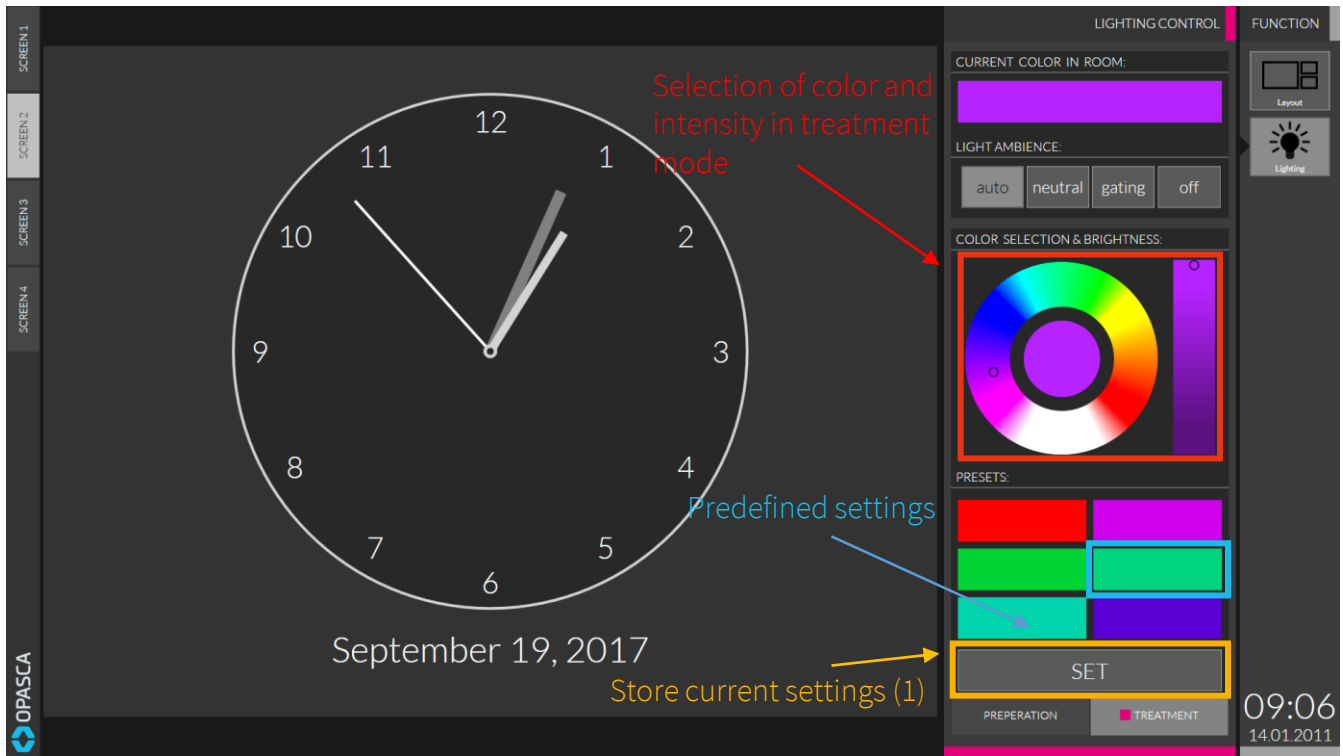

(1) Brightness Setting

To store the current lighting as a preset, use the **SET** button and select a slot from the table of presets above.

Notice:

- Previously stored presets are can be overridden by selecting the corresponding table entry.